

LIPOZERO G39

Professional Cavitation

49 Programs

4 UltraCav39 - 6 CaviMax150 - 10 MF-Plus - 5 Adipose Mobilization NorAd-526 - 4 Man Adiposity - 10 Beauty - 10 Skin

Ultrasonic CAVITATION is the most efficient methodology to fight Cellulite and to eliminate excess fat.

MECHANISM:

- When adipose tissue is submitted to deep low frequency ultrasonic energy, a succession of cyclical compression and decompression which produce cavities containing gases and vapours.
- Thanks to this process, the cavities continue to expand and compress till they explode, destroying in the process the fat cell membrane.
- The fat contained in the adipocytes is freed and eliminated through blood circulation without damaging the surrounding tissues.

THE GLOBUS PROFESSIONAL CAVITATION FREQUENCIES

The cavitation effect is efficiently produced only with low frequencies, lower than 600 KHz, because they are the only one able to penetrate in depth to destroy the adipose tissue. Lower the frequency, deeper is the action of the ultrasonic wave.

- **39 KHz** for adiposity and Cellulite particularly thick and deep that need an strong action (Top Professional)
- **150 KHz** for adiposity and Cellulite less thick and less deep (Professional)
- **526 KHz** to associate the adipose destruction with the production of Noradrenalin (NorAd-526) which helps to burn fat thanks to the increase of the metabolism. Specially used for the adiposity of the men abdomen
- **3 MHz** for superficial beauty treatments and for skin treatments.

MF PLUS - MULTIFREQUENCY TREATMENTS: GLOBUS WORLD FIRST

Scientific researches have demonstrated that the application of Cavitation with Frequency Modulation (alternation of different frequencies) produce better results because the association of different frequencies allows to act at various depth and allows to personalize the treatment according to the type of cellulite.

NEW STYLISH CARRYING CASE: LIPOZERO G39 is supplied with a convenient and modern aluminium carrying case to transport it easily and in a stylish way.



TECHNICAL FEATURES

Power: 3 W/cm²
 Output: 1 - Head: 2 - 1x Ø mm50 & 1 x Ø mm42 - IP65
 Power Supply: main 220V 50/60 Hz or rechargeable batteries (in option)
 Dimensions: 17 x 22 x 6 cm
 Backlight Display
 Emission: continued and pulsed
 Emission Frequency: 3 & 1 MHz, 526, 150 & 39 KHz
 CE Device

EQUIPMENT

1 carrying case
 1 ultrasound Lipozero G39
 1 Multi-frequency Head (3 -1 MHz, 526-150 KHz)
 1 Head 39KHz
 1 contact gel bottle
 1 Power Pack
 1 operating manual



GLOBUS

LIPOZERO G39

Program List

ULTRA-CAV 39 KHz

Total Body 3-6 cm
Total Body > 6 cm
Thigh-gluteus 3-6 cm
Thigh-gluteus >6 cm

CAVI-MAX 150 KHz

Lipolysis
Lipolysis Addominal
Cellulite
Cellulite Oedematous
Cellulite Fibroses
Cellulite with orange skin effects

MF-PLUS

Lipolysis
Cellulite
Cellulite Oedematous
Cellulite Fibroses
Culotte de cheval
Cellulite with orange skin effects
Man Adiposity abdomen
Man Adiposity Hip
Man Adiposity Lower limbs
Man Adiposity Upper limbs

ADIPOSE MOBILISATION NorAd-526

Lipolysis-NorAd-526
Cellulite- NorAd-526
Adiposity - NorAd-526
NorAd-526 1
NorAd-526 2

MAN ADIPOSITY

Man Adiposity abdomen
Man Adiposity Hip
Man Adiposity maschile Lower Limbs
Man Adiposity maschile Lower Limbs

BEAUTY

Toning
Massage
Modelling
Modelling Legs
Modelling abdomen
Drainage
Drainage Legs

Drainage arms
Stretch marks
Active principal application

SKIN

Acne
Face Lifting
Wrinkle Treatment
Scar
Couperose
Body spots
Face spots,
Restructuring skin
Skin smoothing
Photoaging



GLOBUS