

# PRESSCARE G400

PRESSOTHERAPY | 2 OUTPUTS - 6 INFLATION PROGRAMS  
SMART AREA - HEAT

THE FLAGSHIP MODEL OF THE PRESSOTHERAPY LINE, PRESSCARE G400 IS DESIGNED TO IMPROVE CIRCULATION AND PROVIDE A SENSE OF LIGHTNESS AND WELL-BEING TO THE LEGS. DUE TO ITS VERSATILITY AND POWER, THE PRESSCARE G400 IS PARTICULARLY SUITABLE FOR USE IN SPORTS AND FOR THE MOST DEMANDING USERS.

EQUIPPED WITH MULTIPLE INFLATION MODES AND A FOOT-WARMING FUNCTION, IT OFFERS A COMPREHENSIVE, EFFECTIVE AND CUSTOMIZABLE TREATMENT. THE DEVICE COMES WITH SPECIAL LEG SLEEVES FEATURING FIVE OVERLAPPING CHAMBERS FOR UNIFORM COMPRESSION ALONG THE ENTIRE LEG.

## TECHNICAL SPECIFICATIONS

### Outputs

2

### Programs

- 6 inflation modes
- 14 "Smart Area" inflation combinations
- 2 selectable heat settings

### Pressure

40-220 mmHg (in steps of 10)

### Timer

Adjustable from 10 to 60 minutes (in steps of 5)

### Power supply

- Mains
- Lithium battery

### Dimensions (LxWxH)

228x155x87 mm

### Weight

1,2 Kg

## STANDARD EQUIPMENT

- 1 PRESSCARE G400 device - battery included
- 2 Leg sleeves, sizes S/M/L
- 1 Multi-purpose bag
- 1 Power supply
- 1 User manual



## PRODUCT VARIANTS

- G7954 PRESSCARE G400 - S
- G7955 PRESSCARE G400 - M
- G7956 PRESSCARE G400 - L



**GLOBUS**  
ITALIAN EXCELLENCE

# PRESSCARE G400

PRESSOTHERAPY | 2 OUTPUTS - 6 INFLATION PROGRAMS  
SMART AREA - HEAT

Rev. En\_05\_2026

## PROGRAMS DESCRIPTION

### LymphoFlow - LIGHT ACTIVATION (Warm Up)

Progressive inflation of one chamber at a time for a gentle massage that promotes leg relaxation.

**Ideal for:** daily comfort, regaining a feeling of lightness between close training sessions, and a revitalizing massage before a competition.



### PulseTone - POST-WORKOUT RECOVERY

Progressive compression of one or two chambers at a time for a comfortable stimulation. Helps restore energy and tone.

**Ideal for:** post-workout recovery, effective yet comfortable.



### FlowSculpt - DEEP MASSAGE

Sequential inflation of all chambers followed by simultaneous deflation. The compression is more intense and enveloping, designed for a deeper action and a reshaping effect.

**Ideal for:** a more intense and deep massage.



### AirSync - COMBINED SEQUENCE

Combines the benefits of the LymphoFlow and FlowSculpt programs, alternating between gentle stimulation and deep compression.

**Ideal for:** general well-being and moments of relaxation.



### Calf - CALF FOCUS

Concentrated massage between the foot and calf, with targeted compressions to provide an immediate sensation of relief and lightness.

**Ideal for:** relaxation and comfort after daily activities, localized treatment on the calf.



### Thigh - THIGH FOCUS

Treatment focused on the thigh with progressive compression of one or two chambers at a time, for a pleasant sensation of lightness and firmness.

**Ideal for:** a feeling of comfort and localized treatment on the thigh.



### Smart Area

The Smart Area function allows selective inflation of one or more chambers simultaneously, for a fully customized treatment. 14 different chamber selection combinations are possible.

## KEY STRENGTHS



Portable



Compact



5 Chambers



Overlapping chambers



6 Preset programs



Area

Smart area compression



Integrated foot warming



2 Heating levels



19 Intensity levels



Pressure range  
40-220 mmHg



Personalized timing



Mains power &  
Rechargeable battery

**GLOBUS**  
ITALIAN EXCELLENCE

# PRESSCARE G400


PRESSOTHERAPY | 2 OUTPUTS - 6 INFLATION PROGRAMS  
SMART AREA - HEAT

## ACCESSORIES DIMENSIONS

**LEG SLEEVES:** 5-chamber overlapping accessories, ambidextrous, with integrated connector, available in three sizes S/M/L.

**A - GROIN-TO-HEEL** length  
**B - THIGH** circumference  
**C - ANKLE** circumference


| SIZE            | A (cm) | B (cm) | C (cm) |
|-----------------|--------|--------|--------|
| "S" LEG SLEEVES | 76     | 76     | 47     |
| "M" LEG SLEEVES | 87     | 80     | 47     |
| "L" LEG SLEEVES | 96     | 86     | 50     |

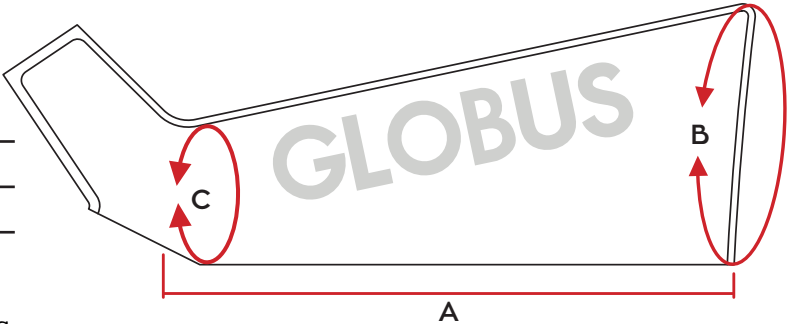

**LEG SLEEVE EXTENSION:** Extensions for the leg sleeves are available in three sizes (S/M/L) to increase the fit by 7 cm at the ankle and 9 cm at the thigh.

**ABDOMINAL BELT:** 5-chamber accessory for treatments on the abdomen, hips, and buttocks.

| ONE SIZE | A (cm) | B (cm) |
|----------|--------|--------|
|          | 125    | 35     |

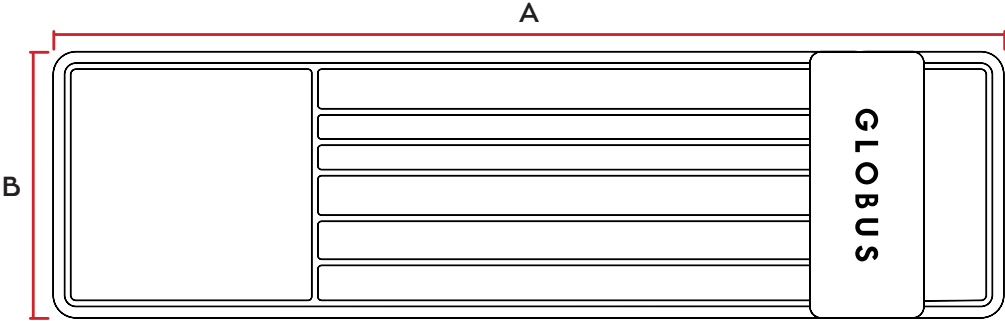

**ABDOMINAL BAND EXTENSION:** Thanks to the Velcro, it can be attached to the abdominal band to improve its fit. Dimensions 40x35 cm.